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USER MANUAL
LOVI Expert 3D Pro Electric Breast Pump

Please read this user manual carefully before using the device.

Important safety information:

Failure to observe this user manual and the safety information may result in danger and the device failure. Use the device only as intended and described in this manual. The information contained in this manual is subject to appropriate technical changes.

INTRODUCTION

Congratulations on your purchase of the LOVI Expert 3D Pro Electric Breast Pump! It is a revolutionary breast pump and a real expert in solving lactation problems.

Thanks to a 3D funnel closest to natural feeding, this breast pump is a breakthrough solution. The 3D funnel, unlike classic funnels, not only sucks but also presses the nipple. The funnel surrounds the nipple and presses the breast evenly, working in multiple dimensions simultaneously, just like a baby's mouth, and tongue does. Even and gentle pressure stimulates the release of prolactin, a hormone responsible for milk production, causing the milk to flow out quickly and naturally, without discomfort.

Expert 3D Pro Breast Pump solves and helps to avoid the most common lactation problems, such as:

- nipple pain,
- breast fullness,
- milk stasis,
- mastitis,

and quickly stimulates milk production after C-section.

LOVI Expert 3D Pro Breast Pump is:

- extremely effective – the openwork 3D funnel holder increases the control of the pressure applied on the nipple;
- comfortable – allows you to express milk even in a reclining position, which is comfortable after a C-section;
- two-phase – two phases of expression reflect natural baby's suckling rhythm: stimulation phase imitates fast suckling movements of baby stimulating milk production and let down; expression phase reflects slower, deeper, and efficient suckling.

LOVI Expert 3D Pro Breast Pump is extremely effective during difficult times.

NOTE: LOVI brand recommends breastfeeding as the healthiest feeding method. It is recommended to start using an alternative feeding method only after consulting your medical practitioner, midwife or pharmacist if breastfeeding is impossible or insufficient.

Breast milk is the healthiest and most valuable food for newborns and babies. It is recommended as sole food given to your baby in the first six months of their life. After six months, it should be combined with solid foods.

NOTE! The breast pump is not intended for expressing milk when a hungry child cannot empty the breast themselves, i.e. in all cases when your milk ducts are blocked.

USEFUL TIPS

When should I use a breast pump?

- If possible, wait until your lactation stabilises. Usually, it occurs within 2-4 weeks after birth. Exceptions to this rule:
- Your breasts produce too much milk, and they become swollen and hard (breast fullness). Expressing a small amount of milk before or during feeding can give you a relief and help your baby grasp the nipple.
- Your nipple is inverted because of the long interval between feedings.
- You have concave, flat or damaged nipples (using a breast pump can improve nipple shape).
- You feel overflow between feeding and do not want to wake the baby.
- The baby is sick and does not want to suckle.
- Your breasts produce too little milk, using the breast pump between feedings helps increase milk production.
- You are working, or you are away from home and want to continue breastfeeding, you should regularly express milk using a breast pump to sustain the natural process of lactation.

How often should I express milk?

If you temporarily cannot breastfeed, it is recommended that you express milk using a breast pump at least eight times a day. That will keep your milk production at the appropriate level. In the beginning, it may be difficult for you to express more milk, but after a few days, you will be able to express more. Even with stable lactation, the possibility of expressing milk in one session is slighter (30-60 ml), therefore to collect one portion of milk, you may require several expressing sessions.

BASIC PRODUCT INFORMATION

Breast pump No. 50/070

The device, an eclectic breast pump, hereinafter referred to as the breast pump, is a product designed for breastfeeding mothers to express breast milk. It enables collection of milk for later use, i.e. in the case of separation from the child. The breast pump helps maintain lactation and prepares for your planned absence or random events such as treatment, cracked and sore nipples.

- The device is intended for personal use.
- The device warranty is 24 months.
- Before using, read the user manual and follow the rules of safe use.
- Keep the user manual together with the device.
- The expected life of the product is two years.
- Silicone parts may change their colour permanently. These parts must be replaced after finding discolouration or after the estimated time of use.

Follow the recommendations on the correct use of the device and its accessories. Never use a damaged or incomplete/broken device.

Damaged or worn parts should be replaced immediately. The manufacturer is not liable for damages resulting from inconsistent with this user manual and improper use. Any damages arising from misuse cannot be the basis for complaint or repairs under warranty. Making any repairs or modifications to the product by unauthorised persons results in loss of warranty and may result in risks to health and life.

GENERAL SAFETY RULES

- If you experience breastfeeding problems or feeding your baby causes you pain, you shall seek medical advice before using the breast pump.
- It is best to use the breast pump for the first time under specialist supervision.
- We recommend to read specialist literature and get to know the principles of baby feeding.

NOTE! Insufficient suction may result in a smaller amount of expressed milk, but too high suction can cause soreness during expressing and reduce milk production. If you experience any of these problems, adjust the suction power (see: SUCTION POWER).

- Keep the device clean. Prevent deposition of dust and other contaminants that could affect the performance of the product.
- Taking care of hygiene is essential for the safety of the mother and her baby.
- Parts of the device which cannot be washed should be protected against water and moisture.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in your area of waste disposal service or the product's supplier.

The manufacturer declares that the product complies with European Union directives and harmonised standards. The product has been marked by conformity assessment procedures.

The product meets the directive's requirements concerning electrical equipment designed for use within certain voltage limits (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class II appliance.

WARNING! This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information. Batch code is on the packaging.

PRODUCT MISUSE

- Never use a product with any signs of damage.
- Never use the device during the bath.
- Never use the product in locations exposed to contact with water, for example near a bathtub, washbasin, kitchen sink or shower unit.
- Never use the device if there is possible that you can fall asleep.
- Never connect the device to an AC power adapter outdoors.
- Never use an AC power adapter with a damaged power cord. The damaged AC power adapter can be repaired by the manufacturer's authorised service centre only.
- Never use the product contrary to its purpose and conditions of use.
- Do not resell a used device.

DEVICE SPECIFICATION

Battery supply	Li-ion battery 3,7V, 2200mAh
AC power supply	100-240 V / 50/60Hz
Maximum power	5 W

SET COMPONENTS

Breast pump parts, fig. 1	
1. Funnel cover	17. Bottle adapter
2. Silicone massaging cushion (also called funnel)	18 Hood/cup with scale
3. Funnel holder	19. Sealing disc
4. Silicone membrane	20. Storing case
5. Breast pump body	
6. Cover	Display, fig. 2
7. Tube connectors 4 pcs	1. LCD display
8. AC power adapter	2. Battery level
9. USB charging cable	3. Working time
10. Tube connecting pump body with the unit 2 pcs	4. Vacuum reduction button
11. Pump unit	5. Vacuum increase button
12. Anti-return valve 3 pcs	6. Phase indicator – stimulation or expression
13. Bottle stand	7. Mute button
14. Bottle 1 pc	8. Suction power level
15. Screw ring	9. ON/OFF pause switch
16. Teat	10. Mode indicator – stimulation or expression

CLEANING AND DISINFECTION INSTRUCTIONS

Before first and each next use disassemble the breast pump. Wash all components of breast pump and bottle in warm water with mild detergent, then rinse thoroughly and disinfect in accordance to rules below.

Before first use, disinfect all components which come in contact with mother's milk (nr. 2, 4, 5, 12, 14, 15, 16, 17, 18, 19) by placing them for 5 minutes in boiling water. This is to ensure hygiene. NOTE! Do not leave components on the bottom of the pot. Stir the water during disinfecting to prevent damage to components.

- Be sure that all components are completely submerged in boiling water during disinfection.
- Do not boil elements that don't have direct contact with mothers milk and electric elements.
- Remove all components from water immediately after boiling.

Before subsequent use, disinfect all above mentioned elements of breast pump and bottle by pouring with boiling water or placing in electric or microwave steam sterilizer, to ensure hygiene.

WARNING! There are no other elements that can be disinfected in microwave sterilizer.

- Do not pour water on bladder cap (nr.6). If you will notice water on or in the cap, pull out the cup plug and leave both elements to dry, so that water won't enter the tubing and the pump unit.
- Warning! Do not disinfect silicone tubing with connectors, plastic brace, bladder cap and electric elements. Do not submerge these elements in water. Wipe with disinfectant.

After each disinfection completely air- or towel-dry before reassembly and next use. Dried parts should be stored in an airtight container or a plastic bag. You can also use a clean towel.


Warning! Partially dry components may affect the operation of your Electronic Breast Pump. Make sure that all parts are completely dry before use.

Washing instructions:

- Always wash the product immediately after use to remove any residual milk.
- Disconnect the tubing with connectors. These elements don't need cleaning, because the silicone bladder protects them from contact with feed.
- Clean all components of the breast pump in accordance with the instructions mentioned above.
- DO NOT use any solvents or chemicals to clean the product.
- Rinse with clean, hot water.
- If you use electric sterilizer for disinfecting elements please follow the instructions given by its manufacturer.

Anti-return valve should be washed very gently and the food should not be allowed to dry out on its surface.

circular bottom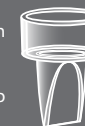

Caution! Always disassemble the anti-return valve by grasping its round base with two fingers. Be careful not to grasp the valve by its tip because it may damage the structure and weaken its suction strength.

tip

Warning! Before each use, check the condition of all elements for cracks, tears, discoloration or deterioration, as they may affect the effectiveness of the device.

How to disinfect the teat?


Before first and each use wash all parts of the product and disinfect. Place for 5 min in boiling water. This is to ensure hygiene. Clean before each use. Boiling and hard water may create lime scale effect. Wash all parts thoroughly with clean water, including areas that may be invisible but where food has passed, to remove any food residues. After using, remove the remaining food from the bottle and the teat, rinse, wash and disinfect in a steam/electric sterilizer or in boiling water. Bottle can be washed in dishwasher (max. temp. 60°C). It is not recommended to place the teat in a dishwasher. Hard chemicals / detergents may damage the product. Do not use any antibacterial agents. Inspect the feeding teat before each use and pull the feeding teat in all directions. Especially when the baby has teeth. Throw away at the first signs of damage or weakness. Do not tighten up the bottle locking ring, as it may disturb the teat's ventilation. Always check for leakage. Never enlarge the teat flow holes by any means, as it can damage the teat. Take particular care when heating in a microwave. Always stir heated food to ensure even distribution of temperature and check liquid temperature before giving it to the child. When warming food in a microwave, remember to leave the bottle open. Do not place product in hot oven or heat in a gas/electric cooker. Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. It is recommended to replace a teat max. after 2 months of use. Retain the instruction for later use. Bottle made from Polypropylene. Batch code is placed on the packaging.

For your child's safety and health


WARNING!

Continuous and prolonged sucking of fluids will cause tooth decay.
 Always check food temperature before feeding.
 Throw away at the first signs of damage or weakness.
 Keep components not in use out of the reach of children.
 Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
 Never use feeding teats as a soother.
 Always use this product with adult supervision.


ASSEMBLING LOVI ELECTRIC BREAST PUMP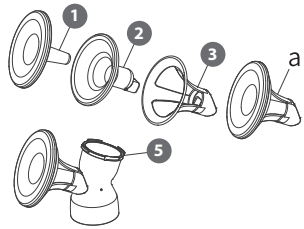


Place the silicone massaging cushion (2) in the funnel holder (3). Then assemble the funnel cover (1) to obtain the element „a“. Important! Make sure that the edges of cushion fit evenly to the edges of funnel holder. Take a note that both elements are profiled and should be properly matched to each other. Connect element „a“ to the breast pump body (5).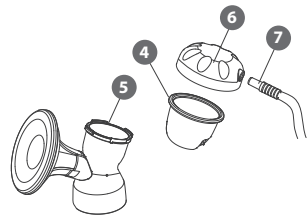


Place the silicone membrane (4) in the breast pump body (5), cover it with the cover (6). Screw the cover onto the pump body tight. Connect the tube connector (7). Press down, making sure it fits tightly.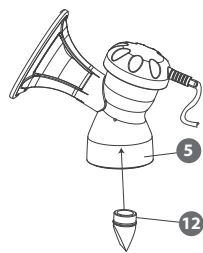


Place the anti-return valve (12) in the breast pump body (5). Press down, making sure it fits tightly.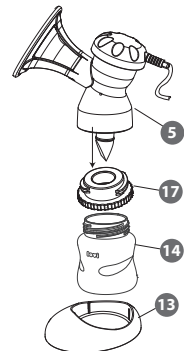

Screw the bottle (14) with the bottle adapter (17) onto the breast pump body (5). Place a complete set in the bottle stand (13). Remove the plastic cover of the funnel.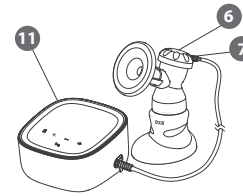

The tube has 2 connectors at the ends. Connect one end to the tube connector (7) located in the cover (6) and the other end to the pump unit (11). Make sure they are connected tightly.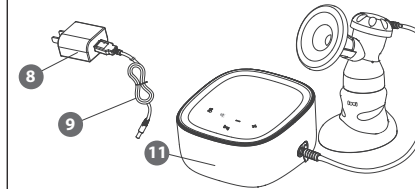


Connect the USB charging cable (9) to the breast pump unit (11) and the AC power adapter (8). The LEDs on display will come on and off, which means that the breast pump is in the standby mode, and the device will start charging the battery.

SOUND WARNINGS

1. Each time any button is pressed, the pump unit makes a beep sound.
2. When the battery runs out, you will hear a beep and the breast pump will turn off after 10 seconds.
3. If you press the mute button (7), you will turn the sounds on or off.

POWER OPTIONS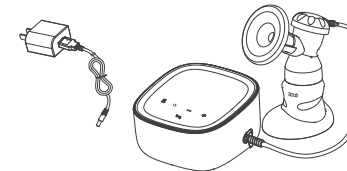

Using AC adapter – place AC adapter ending in the port placed on the pump unit and then plug the AC adapter into an electrical outlet.

The breast pump comes with a 5V 1A power adapter. AC power adapter pins may vary depending on the country of destination. It is recommended to use the original AC power adapter that was included with the breast pump. However, most mobile phone chargers are also 5V 1A or 5V 2A. They can be connected via the

USB charging cable. If the adapter is more than 5V it may damage the product. Important! Before using a different adapter, check its voltage. The above indication is for countries where the EU adapter (supplied) does not work.

Using Li-ion battery.

Charging the battery is done using the AC power adapter and takes about 3 hours.

NOTE! Before first use, fully charge the battery. When fully charged, the breast pump can run about 2/3 cycles. To extend the life of the breast pump, do not let the battery run down fully. When the battery powers the pump unit, the display shows the current charge status. The current battery charge status is displayed in the dynamic form. When the battery runs low, you will hear a sound signal, and the icon will flash. A used battery can only be replaced by an authorised manufacturer's service centre.


USER MANUAL

Before each use:

- Wash your hands thoroughly.
- Ensure that the breast pump is pre-washed and dried, if yes, follow the instructions below.
- Ensure that there is no liquid or moisture in the silicone tube. Never switch on the pump if you notice any water in the silicone tube. Warning! Do not mouth blow the silicone tube to remove water.

NOTE! Before expressing check the charge level of the battery.

If the battery is running low, connect the AC adapter to the pump unit and let the battery recharge.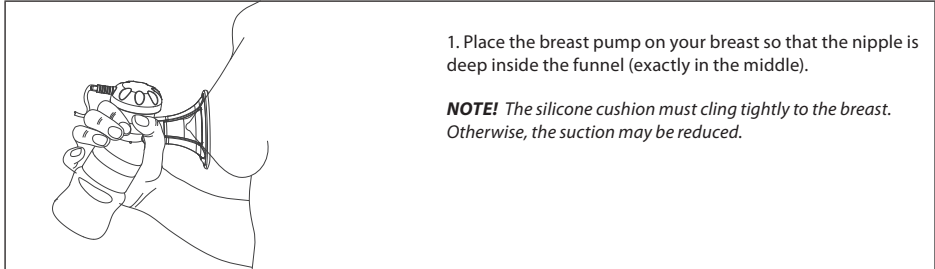

1. Place the breast pump on your breast so that the nipple is deep inside the funnel (exactly in the middle).


NOTE! The silicone cushion must cling tightly to the breast. Otherwise, the suction may be reduced.

- 2. The breast pump is equipped with a touch screen. Touch the on/off icon to start expressing. You will hear a sound signal. You will hear a sound signal during every change of settings on the control panel. (fig. 2).
- 3. At the same time, the pump unit starts counting from 0 to 30 minutes. When the timer reaches 30 minutes, the engine switches off. If you touch on/off icon during expressing, the pump will turn off. The second touch of the on/off icon will start counting down again. The breast pump will switch on and immediately start working in the stimulation phase on the first power level.
- 4. To adjust the suction power in the stimulation phase to your needs, regulate it's level by touching (+) or (-) icons.
- 5. If you want to start the expression phase, press the button 6, fig. 2.
- 6. By touching the (+) or (-) icons on the control panel, you can adjust the suction power in the expression phase to suit your needs. You can return from expression to stimulation phase by touching the same button on the control panel 6, fig 2.
- 7. There are 9 stimulation modes and 9 expression modes.
- 8. If you prefer to skip the stimulation phase or when the milk begins to flow earlier, touch the icon no. 6 (fig. 2) to change the phase.
- 9. If the device was powered from the mains, disconnect the AC power adapter from the socket and turn off the device using the main switch.

NOTE! Before putting the breast pump away from the breast, make sure it is turned off. If necessary, you can put the breast pump away during pumping by placing your finger between the pad and the breast (just as you do it when putting the child away).

WARNING! The breast pump switches off automatically after 30 min. of work. The breast pump switches off irrespective of whether or not it was adjusted during pumping.

MODE SWITCH

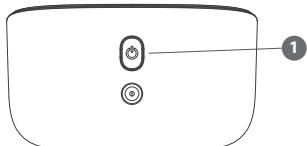After connecting to the power supply, press the button 9 fig. 2 to turn on the device. The breast pump will start running on mode 1 – the default setting. Press the main switch (1) when the device is disconnected from the AC power supply to start operation on battery. Press and hold for a moment the button 9 fig. 2 to start the breast pump operation.

SUCTION POWER

When you turn off the breast pump and restart it, it will start working at mode 1 of the stimulation phase. If milk gets under the silicone massaging cushion, reduce the suction level so that milk keeps up flowing into the bottle.

DECLARATION OF CONFORMITY EU No. 1/2021
In scope of directives: 2014/30/EU, 2014/35/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o.
 Puławska Street 430, 02-884 Warsaw, Poland

We hereby declare under our responsibility that our breast pump (cat. no. 50/070)
brand: Lovi, breast pump - model: XN/MD-2255A,
 is in accordance with the following directives:

- directive relating to electrical equipment designed for use within certain voltage limits
2014/35/EU
- directive relating to electromagnetic compatibility 2014/30/EU
- directive ErP 2009/125/EC
- directive RoHS 2011/65/EU

and is in accordance with the following harmonized standards:

- EN 60335-1:2012 (PN-EN 60335-1:2012)
- EN 55014-1:2006 (PN-EN 55014-1:2017-06)
- EN 55014-2:2015 (PN-EN 55014-2:2015-06)
- EN 61000-3-2:2014 (PN-EN 61000-3-2:2019-04)
- EN 61000-3-3:2013 (PN-EN 61000-3-3:2013-10)
- EN 62233:2008 (PN-EN 62233:2008)

as well as other technician specifications:
 The Commission Regulation (EC) No. 278/2009

The last two digits of the year in which the **CE** marking was affixed: 20

Warsaw, 13.01.2021
(place and date of issue)

Chairman of the Board
 Canpol Spółka z o.o.
 Tomasz Biedrzycki
(name, surname and signature of authorized person, position)

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