bzmi

ODI – smartband

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USER MANUAL **EN**

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linkiem: www.bemi-electronics.com/certificates

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Producent: Shenzhen Minrek Electronic Ltd.

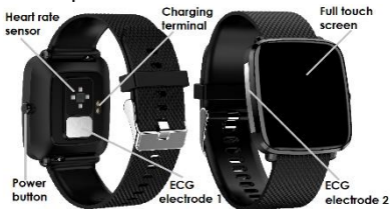
Adres: C602, 6th Floor, Building C, Hangcheng Road, Huafeng International Robot Industrial Zone, Xi'xiang, Bao'an, Shenzhen, Chiny 518126

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USER MANUAL EN

Extended user guide available on www.bemi-electronics.com

Product description

Smart band mode: ODI, in manual called “device”

It is mandatory to read this manual before start to using the device.

To **turn on the device**, press the power button for 5 seconds or connect to the charger. To turn off the device, press the power button for 5 seconds.

Maintaining Equipment

Clean the device regularly, especially part that close to the skin, to keep it dry. Adjust the tightness of the device belt to ensure air circulation. On wrist, where wearing device should not use excessive skin care products. If you have skin allergy or discomfort, please stop wearing device. Do not wear the device during a hot bath or shower. Do not expose the device to cleaning agents, caustic agents or other chemicals. Do not expose the device to high (over + 50 ° C), low (below -10 ° C) temperatures or heat due to, for example, sunlight or other heat source. It may damage the device. The product is not suitable for children and infants. **Note:** IP68 means that this device is

resistant to a single immersion in standing water to a depth of 1.1 meters for up to 1 hour. Exposing the device to tougher conditions can damage it. This device is not intended for water sports, diving and swimming.

Charging

Before start using the device, please make sure the battery is charged. For charging, please use the provided magnet charging cable. The recommended power adaptor current is 5V, 1A. It takes about 2 hours to fully charge device, please follow the steps below to the charge.

1.Place the connector of the charging cable under the device so that its pins contact the metal points in the device charging terminal.

2.Magnetic connector will place itself in right position

3.Connect the charging cable to the power source using the USB connector.

4.Charging animation will display on the device screen

Note: if the device discharged completely then animation of charging may appear after a few minutes.

Downloading APP for Phone and pair with Device

Note: mobile phone must support Android 4.4 or IOs 8.0 or above and Bluetooth 4.0 or above. Some application functions may be invisible in older OS versions of the smartphone. Make sure you have the latest version of your phone's operation system. Scan QT code with mobile phone to download and install APP or search, download and install "H Band" APP from App store or Google Play Store on your smart phone.

While pairing, the phone should be connected to the Internet.

Make sure that the smartphone is set to "Visible to all nearby Bluetooth devices" when pairing. Pairing the device with the phone via APP:

1. Activate Bluetooth on your phone
2. Open the application and click "Click to connect device"
3. Click on the name of your device in the device list. If the connection is successful, app will ask to synchronize data from the device.

Bluetooth connection between phone and device may disconnect in some circumstances. The most common of them are: phone and device are too far from each other (more than 10 meters), there are some walls between them, there are some magnetic or radio interference in environment, user close the APP working in background, user click on the one-button cleaning of the phone.

Introduction on the Function of Device

The main screen menu interface is switched, such as the interface shown above. Slide left or right, up or down to change the screen. Click the screen or icon of function you choose to use it or open function menu. When you are in function slide left to back to previous screen. When the device and APP are first time connected and paired, the time and date will be synchronized with the mobile phone.

Introduction to Function available for Device Interface

NOTE: During test of ECG, Heart rate, Blood pressure and Blood oxygen tested person should be kept still. Any action during the test like eating, speaking, etc. will affect the accuracy of the measurement. Factors affecting the lowering of the accuracy of

measurement include: tattoos, scars, moisture, thickness, strong pigment and hair of the skin in the place of measurement. Keep the heart rate sensor clean and close to the skin. If you wear it too loosely, the sensor's light will be exposed, resulting in inaccurate measurement values. Do not look directly at the sensor green light. During test, please keep your body inactive and sit still, otherwise the time of test will be longer or data will be inaccurate. If there is a big difference in the measured data, please take a break and test again. Longtime measurement may lead to overheating of the device. In case of overheating, please remove device until it cools down. The cold weather will affect your measurement results, please measure while keeping warm. Place the device on the wrist correctly and enter the heart rate menu to measure the heart rate. The data of ECG, Heart rate, blood pressure and blood oxygen measurement are for reference only and cannot be used as a basis for medical related aspects.

1.Main screen (clock face)

There is few clock faces to choose. To change clock face, on main screen slides down or up.

A-hour, B-minute, C-date, D-steps, E-distance, F- calories burned, G- battery level, H-Bluetooth connection:

Connected

Not connected

2.Step - Steps, calories and distance will count automatically after start of using the device. The data is displayed every specific number of steps, their amount depends on distance already walked and device software version (between 1-25 steps). These data are approximate due to individual differences between users. In order for the steps to be recorded, the hands should move in a natural manner when walking, for example

walking with a stroller may not be read as a step. More data and settings is available in the APP.

3.Sleep – Sleep monitoring automatically recording your sleep started any time between 18:00 to 08:00 the next day. Shortened data is available in the device. More data is available in the APP. In APP can also set up sleeping time goal. So this data can be recorder first need to turn on this option in APP Settings: Mine>My device>Switch settings>Scientific sleep

4.Sport

4.1.Enter sport mode. Then find the right kind of activity by slides down and select by click the screen to enter.

4.2.Sliding the screen to down will jump to other data interfaces and interfaces where you can Pause or End recording.

4.3.When the activity time is longer than 1 minute, the data can be saved.

History of record and more data is available in the APP.

5.ECG - ECG test should be activated from application. When activated from device only heart rate will be showed. Its alternative way to check Heart rate, instead of Led light, in this test to measure the Heart rate are used electrodes. Information How to do this test is located in this manual in "Introduction to Function available for APP Interface"> 6.ECG

6.Heart rate - Using Led light to measure heart rate and show status in real time.

7.Blood pressure - The result of the test will be one measurement score.

8.Stopwatch - Click the stopwatch to enter it interface. Click the start icon to start time measurement.

9.Counting down - After setting the time, press start button to counting down.

10.Information (Message) - The message interface allows to view on device messages came on phone as SMS, incoming calls and chat software. Bluetooth connection phone-device is required for this function. Note: Notification settings are available in APP. Regardless of the settings in the APP, if the

notification does not appear as a notification in the phone, it will not be sent to the device. Check the phone's notification settings

11.SpO2 (Blood oxygen) - oxygen levels in the blood in real time.

12.Find phone - When the device is connected to APP, clicks on Find phone, and the phone will pop up with a ring tone.

13.Setting

13.1.Information - selection, which of your phone application will show notification in your device.

13.2.Brightness - setting the brightness of the screen

13.3.Auto-Lock - setting the time of the automatic turning off the screen

13.4.Switch - Switching on/off additional functions of device.

14.Music - When the mobile phone is connected to the device, music playing/pausing and song switching in the mobile phone music player can be controlled by device.

Introduction to Function available for APP Interface

NOTE: first connect device with APP

You can send out the data contained on the page you are viewing by pressing the button  and choosing the communicator.

Pressing start button  will start recording.

Pressing  located next to the data will open page with this data description.

Pressing  will turn on or off this function.

This look  means that currently function is turned on.

This look  means that currently function is turned off.

APP have main three pages: Dashboard, Running, Settings

Dashboard

Running

Settings

DASHBOARD

1.Test Menu - In Dashboard click "More" to open test Menu. You can choose and initiate Heart rate, blood pressure, Oxygen in blood and ECG test by clicking its icon.

2.Sport - Showing number of steps, distance and calories in total and in time periods, done today and previous days.

3.Sleep - Showing the quality and data of sleep. So this data can be recorder first need to turn on this option in Settings: Mine>My device>Switch settings>Scientific sleep

4.Heart rate - Showing the history of heart rate status. Data are shown in periods of 30 minutes. If there is no data in the some field, it means that there was no test at that time.

5.Blood pressure - Showing the history of blood pressure status. Data are shown in periods. If there is no data in the some field, it means that there was no test at that time.

6.ECG - To do the test we suggest lie down or sit in comfortable position. Test can be done in many position, our suggestion is: (If you wear the device on your left hand) put your left hand on your right breast. After starting the test, put one of the right hand fingers to the "electrode2"

6.1.In APP dashboard, slides down to ECG section and click on it.

6.2.Click on "start testing"

6.3.Click Play button

6.4.When your device on one hand, put your other hand finger on "electrode 2" on device side

6.5.Test is running, on the phone screen you can see percent of progress, and result in real time.

6.6.When test is finished choose to save the record or not.

6.7.Test records can be viewed in APP in ECG first interface.

6.8.You can also send test record by chosen communicators.

NOTE: During the test both electrodes should stick to the skin. More skin area will stick to electrode the more accurate test will be. Do not press to hard. Because to the use of touch screens,

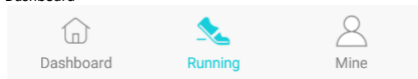
the skin of some fingers may be thicker, use your finger with a thinner skin for the test.

7.Blood Oxygen - Oxygen level in blood during a sleep.

8.HRV - Showing line graph of heart rate change during a sleep.

9.Module Manager - Allow to manage what can be seen on Dashboard


RUNNING

Contains the history of sport activity recorded with sport mode and allow to start sport mode from APP side.

GPS Movement – using GPS module in mobile phone to mark the route of activity on the map. This function no require to use smart band or smart watch. Allow to view of previous GPS movement records. Press play button to start recording.

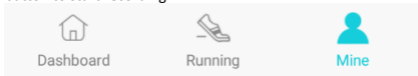Smartwatch Movement – Using device to record the data of activity. Allow to view of previous sport mode records. Press play button to start recording.

MINE

Contains settings about user, device and APP.

1. [Android](#) > name of user. Press the name to change it.

2. My Device. Name of device, press it do go inside the device menu  BR15

2.01.Message notification – select the application/function of mobile phone that will sent notification to your device.

2.02.Alarm settings - When device is connected to the APP, the alarm clock set inside the APP can be synchronized to the device. Clicking on the icon on the device screen will stop the vibration alert.

2.03.Sedentary setting – “start time” and “stop time” set up the time between which this function will be activated. Reminding interval set up the time of no moving extending which will activate the reminder.

2.04.Heart rate alert – set up upper limit of resting heart rate exceeding which activates the alarm.

2.05.Turn wrist detection – automatically wakes up the screen when you raise your hand with device, to the chest height while rotating the screen towards the face.

Set up “Start time” and “Stop time” so screen of device will not light accidentally wake up when you don’t want it, for example when you sleeping. Set up “Sensitivity” of activation to your private preference.

2.06.Pair device – synchronize connected device with mobile phone

2.07.BP private mode – set up proper blood pressure level according your personal preference.

2.08.Screen-On time – set up the time of disuse after which the device display will automatically turn off

2.09. Switching settings:

2.09.1.BP automatic monitoring – automatically turn on blood pressure, in at least ones in hour.

2.09.2.Blood oxygen nighttime monitoring – monitoring level of oxygen in blood when you sleeping in the night.

2.09.3Disconnect alert – inform when device and mobile phone become disconnected.

2.09.4.Scientific sleep - sleep quality monitoring.

2.10.Take a photo – turn on mobile phone camera, picture can be taken by pressing screen of device.

2.11.Reset device password – set up or change device Bluetooth connection password.

2.12.Firmware update – allows to update firmware of device to last version

2.13.Clean Data – delete all additional data and restore to factory settings

2.14. Disconnect – disconnect Bluetooth connection between smartphone and device

3.Steps goal – set up how many steps you want to do in one day. When you reach this goal, this achievement will be displayed on device.

4.Sleep set up how many hours you plan to sleep each night.

5.Unit settings – set up is distance will be showed in kilometers (metric) or in miles (Imperial).

6.Change theme – set up color of APP.

7.About us – showing version of APP and possibility of update.

8.Help - answers to potential problems during use.

9.Log out – will log out the user from the application.

Warning: Although the smart device can monitor health condition line: real-time dynamic heart rate, ECG, Blood pressure, it cannot be used for any medical purpose.

Basic Parameters: Screen type: 1,3 inches TFT; battery capacity: 220 mAh; protection level: IP68

Proper disposal of this product. The device cannot be disposed of together with household waste. To prevent potential threats to the environment and human health due to improper disposal of waste, it is mandatory to take used electronic equipment to a designated collection point, or to the place

where the product was purchased for the purpose of recycling electronic waste. **Warning:** Lithium battery inside.

This device, smart band model ODI, is in compliance with Directive 2014/53/EU of the European Parliament and of the Council. The complete declaration of conformity is available

online at the following link: www.bemi-electronics.com/certificates

This manual is not covered by the warranty, and the appearance and/or functionality of the software installed on the device may

be different from the one described. The Guarantor reserves the right to modify the instruction manual at any time without prior notice.

Maximum power consumption - 0.264W; Standard power consumption - 0.119W; Power consumption in standby mode - 0.182mW; Operating Frequency Range of device - 2,4 GHz

Manufacturer: Shenzhen Minrek Electronic Ltd.

Address: C602, 6th Floor, Building C, Hangcheng Road, Huafeng International Robot Industrial Zone, Xi'xiang, Bao'an, Shenzhen, China 518126

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Valmistaja: Shenzhen Minrek Electronic Ltd.